



Airbrush Liposculpture®

*Unleashing the artist
in the surgeon®*

TECHNOLOGY, INC.

BST

**Quick Start
Instructions**

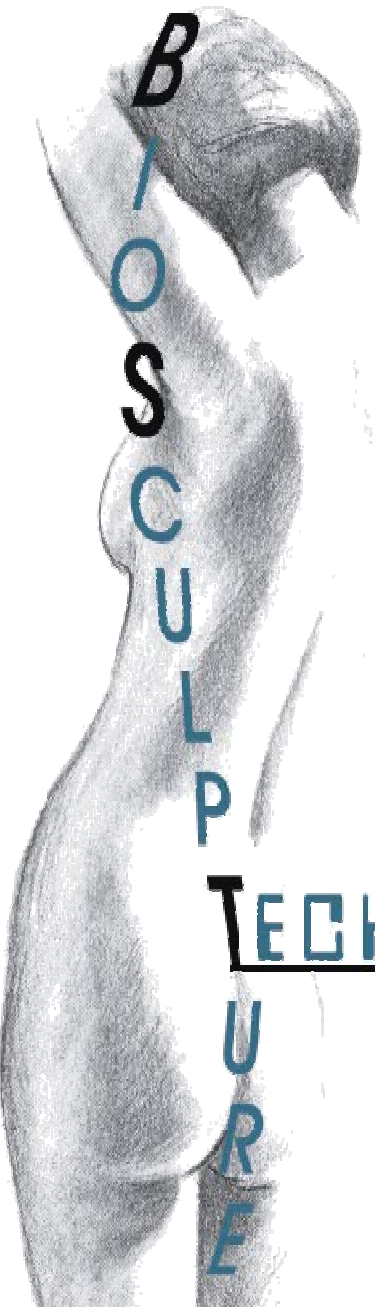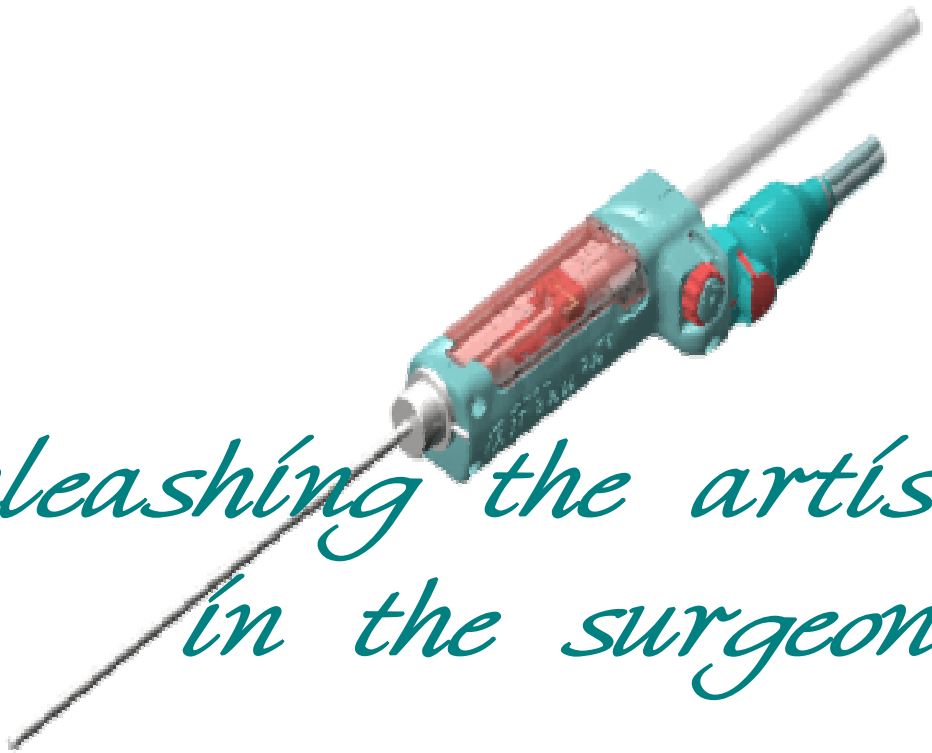

B
/
O
S
C
U
L
P
T
U
R
E

Technique

- As the *Airbrush® Liposculptor* does all the work, manual reciprocation is not needed. The optimal technique is that of “continuous, even, slow motion” and feathering. **Avoid “hovering” in one spot and don’t let the outer cannula slot emerge from the incision while the inner cannula is reciprocating.**
- Set **full stroke first and then gradually decrease** to desired length and power for maximal smoothness and silence of operation.



- **Locate the slot superficial to the area of fatty excess** and allow the *Airbrush® Liposculptor* to effortlessly remove fat in a precise and controlled fashion until the desired effect is obtained.
- For enhanced results you may **briefly direct the slot superficially to encourage skin retraction.**

Unleashing the Artist in the Surgeon®

- Since the concern is only the proper placement of the instrument, judgment of how much fat to remove from each location, and an even feathering of the borders, **the surgeon is free to be an artist** without the drudgery of manual cannula reciprocation.
- The surgeon can **pinch the pannus** about the cannula and literally feed fat globules into the slot for aspiration of any residual unevenness or lumps under precise control.



- In addition to selecting an appropriate sized cannulas set, to increase the control of fat removal the **surgeon may adjust the stroke or rate** (*Airbrush® Liposculptor I* only) **of reciprocation** using the knob(s) on the hand piece. **Turn stroke fully off and then fully on to reset.**

Unleashing the Artist in the Surgeon®

- Though most surgeons will prefer the default amount of stroke awareness we set at the factory, **the cushion may be adjusted** to the surgeon's preference by the manufacturer's representative to be more or less gentle.
- The **reciprocation of the inner cannula is visible** through the transparent chamber cover.



- To eliminate drag, decrease vibration, and increase the speed of reciprocation you may temporarily **fasten the vacuum tubing, with about 3" of slack, to the multicore cable** using a sterile cable tie approximately 2' behind the hand piece if desired.

BioSculpture Technology, Inc.
40 Central Park South
New York, N.Y. 10019
Telephone: (212) 977-5400
Fax: (212) 586-9529

sales@liposculptor.com
www.biosculpturetechnology.com

© 2003-6 BioSculpture Technology, Inc.

Unleashing the Artist in the Surgeon[®]